



Invisible to Everyone But You



TempoCem[®]ID

It stands out by blending in

TempoCemID is the esthetic, temporary cement that is invisible when you want it to be, yet detectable when you need it to be. Formulated for optimal transparency, it will never impact the shade of your provisional, making it an ideal choice for both posterior and anterior restorations alike. Unlike other esthetic temporary cements, excess TempoCemID is readily detectable (even below the gum line) and can often be removed in one piece. Non-eugenol TempoCemID flows and mixes easily, doesn't stick to instruments and is indicated for long-term temporaries.

**Invisible under restorations, yet
detectable for easy excess removal**



See your esthetic worries vanish before your eyes with TempoCemID

FEATURES & BENEFITS

- Beautiful esthetics thanks to optimal transparency
- Easy one-piece removal of excess
- Optimal adhesion with easy removability
- Eugenol free
- Excellent ease of use, no sticking to instruments
- Flows and mixes easily
- Perfect for long-term temporaries



Ordering Information

Item #	Product	Contents
213200	TempoCemID SmartMix	1-5ml Syringe, 10 Smartmix Tips
110728	TempoCemID Smartmix Tips	Bag of 25



From the
makers of
Luxatemp


DMG America, LLC
65 Challenger Road, Suite 340, Ridgefield Park, NJ 07660
tel: 800.662.6383 email: info@dmg-america.com
web: www.dmg-america.com




TempoCemID

Quick Guide


Recommended Use

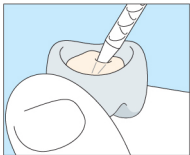


1. Apply TempoCemID to restoration.

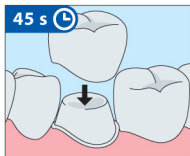


2. Insert restoration within 45 s after start of mixing.

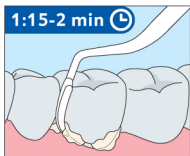


3. Remove cement residues within 1:15–2 min after inserting.

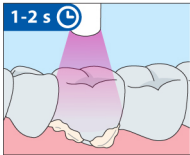

4. Optional: Lightcure cement residues for 1–2 s per side and remove.

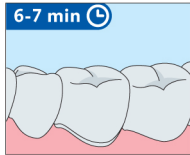

5. Allow material to chemically cure for 6–7 min after start of mixing.

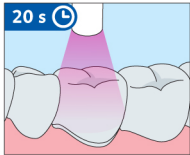

6. Where necessary, light-cure cement for at least 20 s per side.



Please consult the enclosed instructions for use for further information.
DMG America LLC · www.dmg-america.com