

CLEARFIL™ S³ BOND Universal Quick *[CLEARFIL™ TRI-S BOND Universal Quick]*

Quick without Compromise

No Waiting Time with Real Clinical Benefits


Less Technique Sensitive

Less Risk of Contamination




No Waiting Time

The new original MDP & Amide chemistry provides a unique RAPID BOND TECHNOLOGY. Unlike other universal adhesives, there is no need to wait for it to penetrate the dentin before you proceed.

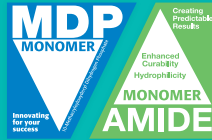

The new **Amide** monomer rapidly permeates dentin & enamel eliminating waiting time.

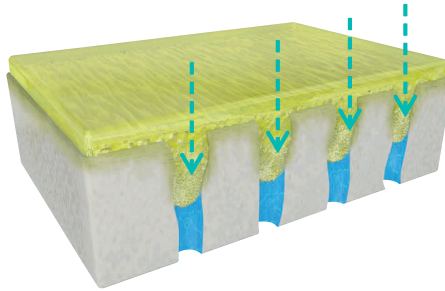

The original **MDP** monomer creates a strong chemical bond to hydroxyapatite. MDP has a proven excellence in Adhesion for over 30 years.

RAPID BOND TECHNOLOGY offers consistent, predictable results with all etching procedures & virtually any application time.




REAL CLINICAL BENEFITS:

Less Technique Sensitivity
Less Risk of Contamination
More Efficiency

Quick without Compromise: Excellent Bond Strength


Self-Etch Shear Dentin Bond Strengths (Notched Edge, MPa)

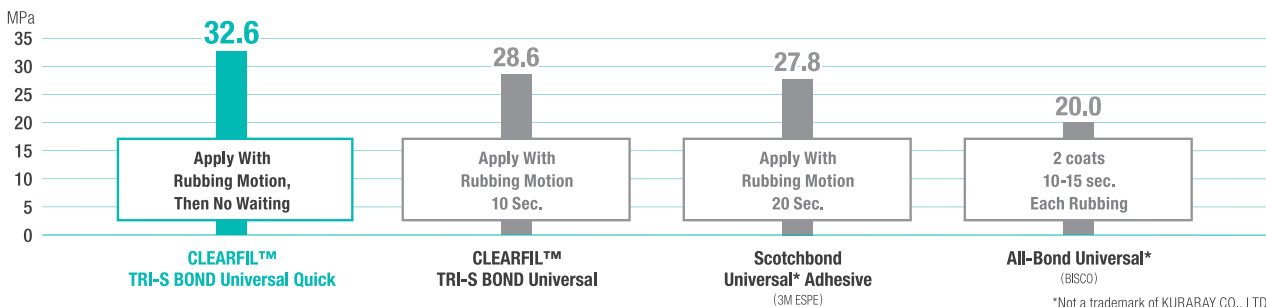

*Not a trademark of KURARAY CO., LTD
Data source: Kuraray Noritake Dental Inc.

Excellent Bond Strength

Shear Bond Strength

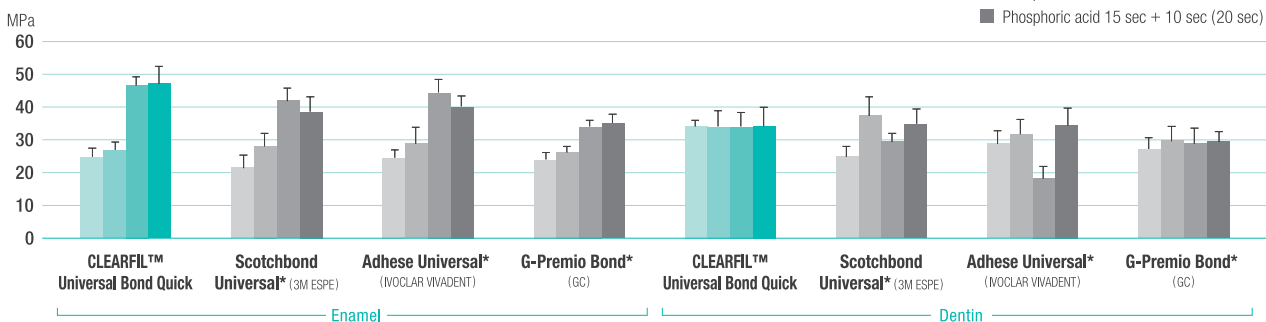

Prof. Dr. Miyazaki et al. 2019 IADR/AADR/CADR General Session & Exhibition
Bond Performance of Universal Adhesives with Different Application Times
*Not a trademark of KURARAY CO., LTD

Improved Bottle Delivery

- **One hand** for opening, dispensing and closing
- **Unique Nozzle** for drop control
- **Clean after use**
- **Minimal waste**
- **Compact**



A Truly Universal Multi-purpose Adhesive

All Etching Techniques



Self-etch
no phosphoric acid

Selective-etch
phosphoric acid
on enamel

Total-etch
phosphoric acid on
Enamel and dentin

All Clinical Indications



**Direct
Restoration**

**Post Cementation /
Core Build-Up**

**Cementation
Indirect Restoration**

Intraoral Repair

Virtually no Post-Operative Sensitivity

Low film thickness

Allowing esthetic and good fit of indirect restorations

Best Results in Gordon J. Christensen Clinicians Report

“CLEARFIL™ Universal Bond Quick had best overall results, most notably for its superior bond strength values and best ease of application”

Vol. 11, Issue 5, May 2018

More Features Than Leading Universal and Total-Etch Adhesives

Universal Adhesives

	CLEARFIL™ TRI-S BOND Universal Quick	Scotchbond Universal Adhesive* (3M ESPE)	ALL-BOND Universal* (BISCO)
Fluoride Release	Yes	No	No
Bonds Zirconia	Yes	Yes	Yes
Contains Silane (Bonds e.max* & Porcelain)	Yes	Yes	No
Economic Bottle Packaging	Yes (3 Bottles)	No	No

Total-Etch Adhesives

	CLEARFIL™ TRI-S BOND Universal Quick	Optibond Solo Plus* (KERR)	Prime & Bond® NT* (Dentsply Sirona)
Indicated for All Etching Techniques	Yes	No	No
May Be Used as Self-Cure Adhesive	Yes ¹	No	Yes ^{**}
Bonds Zirconia	Yes	No	No
Bonds Lithium Disilicate	Yes	No	No

*All information from mfg. Instructions For Use. Not a trademark of KURARAY CO., LTD. **Must purchase a separate dual-cure activator
1. Indicated for use in self-cure mode with self-adhesive cement of Kuraray Noritake Dental Inc.

Basic Procedure

Direct Restorations using Light-Cured Composite Resin



Self-etch mode

Move to next step

Selective-etch mode 10 seconds

Rinse & Dry

Total-etch mode 10 seconds

Rinse & Dry



Apply with a rubbing motion



(No waiting time after application)



Dry

5 seconds
use a vacuum aspirator to prevent Bond from scattering



Light-cure


5 seconds
(High performance LED)

Indirect Restorations Cementation of Indirect Restorations with PANA VIA™ SA Luting Multi to Zirconia Crown



Clean and dry

Clean and dry the tooth surface, and trial fit the restoration

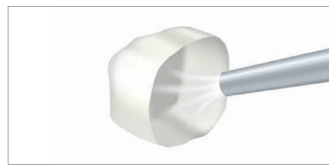

Blast

If necessary, Blast with alumina powder (30-50µm, 0.1-0.4MPa / 15-58PSI / 1.4kgf / cm²) then ultrasonic clean and dry




Tooth Pretreatment

- Self-etching (move to next step)
- Selective-etching
- Total-etching



Apply

Apply CLEARFIL™ Universal Bond Quick with a rubbing motion and dry by blowing mild air

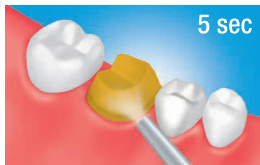

Cementation

Cementation using PANA VIA™ SA Luting Multi according to the manufacturer's instructions



CLEARFIL™ TRI-S Bond Universal Quick Bottle Refill

#4401-WD

Bottle (5 ml)

CLEARFIL™ TRI-S BOND Universal Quick Value Pack

#4402-WD

Bottle (3 × 5 ml)

CLEARFIL™ TRI-S BOND Universal Quick Mini Pack

#4403-WD

Bottle (1 ml)



Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

Website www.kuraraynoritake.com

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

"PANA VIA" and "CLEARFIL" are trademarks of KURARAY CO., LTD